TENSIOATTIVO CONCENTRATO CONCENTRATED SURFACTANT

Senza Sodio Azide / Sodium Azide Free

REF. 3970 – 2x100 ml

IVD

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INTENDED USE

The Surfactant is a solution that can be used in all automated instruments to allow a better sliding of the liquid in the hydraulic circuit of the analyzer and thus prevent the adhesion of small drops of liquid on the walls of the tubes, on the needle, on the reading cuvette.

PRODUCTS CHARACTERISTICS

Solution: Surfactant with antibacterial.

PREPARATION

Use solution in the ratio: 1:1001 (1+1000).

Pour 5 ml of Concentrated Solution in the container of the washing water of the analyzer and add 5 liters of bidistilled water.

You should prepare a weekly wash solution, eliminating the old residual solution and thoroughly rinse the container with distilled water.

In case of black mold in the plumbing system, bring the volume of the solution to 10 ml in 5 liters of distilled water.

STORAGE AND STABILITY

The product is stable until the expiration date indicated on the label when stored at 2-25°C, protected from light and in the absence of contamination. Keep bottles closed when not in use.

PRECAUTIONS AND WARNINGS

Use in accordance with good laboratory practise, avoiding contact with skin, eyes and swallow.

WASTE DISPOSAL

Dispose of product according to local laws.